



Produits Dentaires SA

Rue des Bosquets 18 . 1800 Vevey . Switzerland

GB

DIRECTIONS FOR USE

Zinc Oxide

Co-formative agent of ZOE cement

Composition Zinc Oxide 100 %

Properties

When mixed with Eugenol, Zinc Oxide powder forms a cement (paste) used in dentistry, due to its compatibility with both hard and soft tissues of the mouth as well as to its reported action.

Indications

- > Temporary obturation of the root-canal
- > Protection of nearly exposed pulp
- > Periodontal dressing

Direction for use

On a glass mixing slab, mix a few drops of Eugenol to the powder of Zinc Oxide into a paste of the required consistency.

Setting time

In presence of moisture, the mixture will begin to set (slowly). Higher moisture rate or temperature will increase the setting reaction.

Storage

Store at room temperature between 5 and 30°C, in a dry place

Packaging

	REF
Bottle of 100 g	11431
Bottle of 250 g	11432
Bottle of 500 g	11430
Bottle of 1000 g	11433



> FOR DENTAL PROFESSIONAL USE

compatibilité avec les tissus buccaux durs et mous ainsi que de son action décrite dans la littérature.

Indications

- > Obturation canalaire temporaire
- > Protection de la pulpe exposée
- > Pansement parodontal

Mode d'emploi

Mélanger extemporanément, sur une plaque de verre, quelques gouttes d'eugénol à la quantité nécessaire de poudre d'oxyde de zinc, de manière à obtenir une pâte de la consistance désirée.

Temps de prise

La prise du mélange commence en présence d'humidité. Une augmentation du taux d'humidité ou de la température accélère la prise.

Stockage

Conserver à température ambiante entre 5 et 30°C, dans un endroit sec.

Présentation

	REF
Flacon de 100 g	11431
Flacon de 250 g	11432
Flacon de 500 g	11430
Flacon de 1000 g	11433



> POUR USAGE PROFESSIONNEL DENTAIRE

RU

УКАЗАНИЯ ПО ПРИМЕНЕНИЮ

Zinc Oxide

Оксид цинка

При смешивании порошка с эвгенолом образует «временную» цемент-пасту.

Показания

- > Временное пломбирование корневых каналов;
- > Защита поврежденной пульпы

Упаковка

100 г.

Код

11431



> ДЛЯ ПРОФЕССИОНАЛЬНОГО ПРИМЕНЕНИЯ В СТОМАТОЛОГИИ

F

MODE D'EMPLOI

Oxyde de Zinc

Co-formateur d'un ciment ZOE

Composition Oxyde de zinc 100 %

Propriétés

Mélangé à de l'eugénol, l'oxyde de zinc forme un ciment (pâte) utilisé dans l'art dentaire du fait de sa