«Cavity Liner» Paste, Calcium Hydroxide Paste

REF 11020

GB DIRECTIONS FOR USE

Radioopaque Calcium Hydroxide Paste for lining and capping

Composition

1 ml of paste contains 540 mg Calcium Hydroxide and 70 mg Barium Sulfate in a Methylcellulose base

Properties

«Cavity Liner» Calcium Hydroxide Paste is a creamy paste which can be used as a cavity liner or a temporary dressing. Owing to its alkaline pH (approximately 12), Calcium Hydroxide, when used as a liner or a pulp capper, neutralizes the free acids of Zinc Phosphate and Silicate Cements. It also helps to reduce the sensitivity of freshly cut dentine and promotes the regeneration of secondary dentine.

Indications

- > As a liner
 - pulp capping on vital pulp (indirect capping)
 - capping after pulpotomy (direct capping)
 - cavity lining, prior to filling
- > As a dressing
 - root-canal treatment and periapical therapy

Directions for use

- > Pulp Capping
 - Make sure of the pulp is vital. Where necessary, treat with an antibiotic
 - Cap with the paste and allow to dry
 - Remove any residue with an excavator
- > Cavity Lining
 - Clean the cavity thoroughly
 - Apply the paste
 - Remove any residue
 - Apply filling material directly on the liner
- > As a temporary dressing
 - Prepare the canal meticulously and fill it with the paste
 - Seal the opening with a cement
 - Where necessary repeat the operation after one month

«Cavity Liner» Calcium Hydroxide Paste is easily removed with an endodontic file and a few drops of water. After each use wash the cap with water and recap the syringe immediately without drying. The applicator tips can also be washed and sterilized.

Storage

Store at room temperature between 15°C and 25°C, protected from direct light

Packaging

Set with 1 dispensing syringe of 2.5 ml
+ 2 applicator tips

- > FOR DENTAL PROFESSIONAL USE
- > MADE IN SWITZERLAND

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